Alginat

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Nettogewicht: 500g

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GB Instructions for use

It does not require special treatment before use. Follow the powder / water ratio. Pour the powder into the water, mix vigorously and work in compliance with the declared times (*), measurements were taken at 23°C with deionized water. When using specific mechanical mixers for alginate-based impression materials, follow the manufacturer's instructions. Variations in temperature and/or water hardness and in the speed and/or friction during mixing cause changes in the working time and in the initial setting time. Wash, rinse, shake and disinfect the impression (specific alcohol-based disinfectants are recommended). By keeping the impression in a sealed plastic bag, without adding anything else, the model can be poured within 100 hours.

WARNINGS: For use, wear protective gloves, eye protection and protective clothing. The powder is "Dust Free", however, caution is recommended to avoid inhalation. In case of contact with eyes wash thoroughly with water. After taking the powder close the bag. Use within the expiry date. Do not use on patients allergic to spearmint flavour. Any possible serious accident related to the product must be reported to LASCOD and to the Competent Authority of the Member State in which the user is established

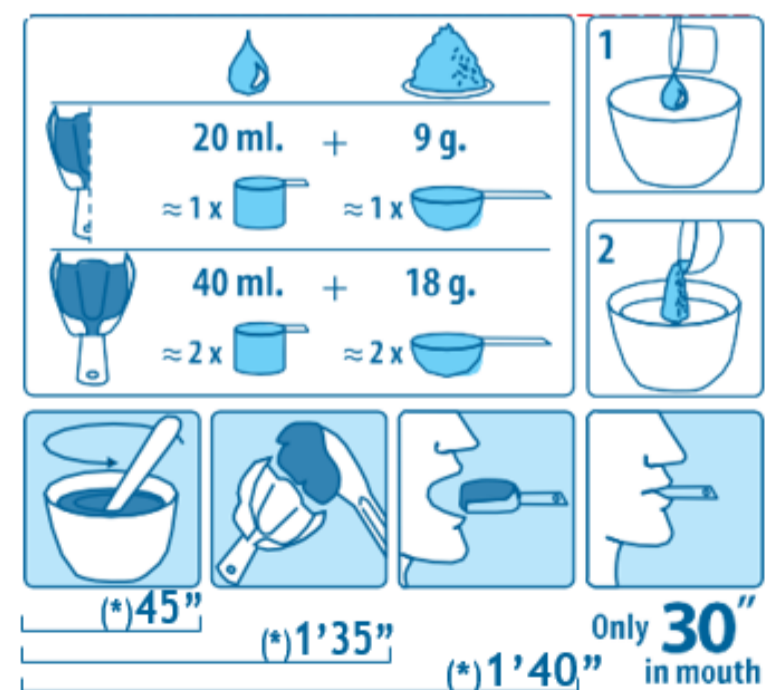
For dental professional use.

Store in cool and dry place.

Dust free Alginate impression material

Color: blue

Flavour: spearmint

(* Measurements taken at 23°C with deionised water.

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Lascod SpA