## Omniwipes wet&easy (neutral, fleur, lemon) Desinfektionstücher/Disinfection wipes

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### GB Instructions for use

**Type of application:** As a medical device: Alcoholic wet wipes for the prophylactic surface disinfection of alcohol-resistant medical devices. Not suitable for the (terminal) disinfection of invasive medical devices. As a biocidal product: Alcoholic wet wipes for the rapid disinfection of alcohol-resistant surfaces.

**Notes on application:** Open the top of the resealable dispenser bag and remove an impregnated wipe. Then close the bag tightly again straight away. Carefully wipe the area to be disinfected and let the disinfectant take effect. Only for use on alcohol-tolerant materials (not suitable for acrylic glass). Once the bag has been opened for the first time, the wipes can only be used for a maximum of 28 days. Before opening the bag for the first time, the opening and shelf life date must be entered into the corresponding field or transferred onto the plaque manually.

**Application times (of the impregnation fluid):** Bacteria and Candida albicans as per EN 13727, EN 13624 and EN 13697 (dirty conditions): Conc. 1 min., TB (tuberculosis) inactivation (M. terrae) as per EN 14348 (dirty conditions): Conc. 1 min., Enveloped viruses (e.g. HBV, HIV, HCV) as per DVV (dirty conditions): Conc. 1 min., "Limited virucidal PLUS" (noro-, rota- and adenoviruses) as per EN 14476 (dirty conditions): Conc. 1 min.

**Application times of the impregnated wipes:** Bacteria and Candida albicans as per EN 16615, VAH (Association of applied Hygiene) under dirty conditions: Conc. 2 min.

**100g impregnating fluid contain:** 43.0g ethanol; 8.0g 1-propanol; 0.055g didecyldimethylammonium chloride.

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**Safety Instructions:** H228 Flammable solid. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear protective gloves / eye protection. Store in a closed container. Keep out of the reach of children. Read label before use. Flash point as per DIN 51755: approx. 27°C. Application max. 50ml per m. The total applied amount per room must not exceed more than 100ml per m of room area. Keep the containers tightly closed. Only put the packaging into the recycling system if it is empty of residue. Dispose of residual product in accordance with EAK 07 06 99. Directive (EC) 648/2004: Contains: fragrances

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BAUA-Nr.: N-72848

UFI:1AFC-JN7C-H10P-K0H8

PRISMAN GmbH

Otto-Hahn-Ring 6-18

D-64653 Lorsch

Tel.: +49 (0) 6251 866980-0

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OMNIDENT Dental-

Handelsgesellschaft mbH

Gutenbergring 5

D-63110 Rodgau Tel.: +49

(0) 6106 874-0